

Made in Italy by



**EMODIAL**



# EXIT-POCKET® PT

Peritoneal Pocket  
for catheter protection



[www.valiantmedical.co.uk](http://www.valiantmedical.co.uk)

+44(0)330 303 9090

[info@valiantmedical.co.uk](mailto:info@valiantmedical.co.uk)



# EXIT-POCKET® PT

## DESCRIPTION

Contains the catheter lumen between each dialysis treatments, protecting the exit-site.

## INDICATIONS

Protects catheter surrounding skin and acts as a barrier against microbial contaminations.

## PURPOSE

- Quick and easy placement.
- Suitable to preserve the skin from the catheter mechanical rubbing.
- Comfortable fit, it can be wear under clothing and does not interfere with the normal daily activities.
- Helps to improve the quality of life of catheterized patients.
- Easy and fast removal.



## REFERENCES

Data and technical specifications can be changed without notice.

CODE	POCKET (cm)		PCS / PACK	PACKS / BOX
	L	H		
S2013/PT	24	8	1	40
S2010/PT	18	8	1	40

## DIRECTIONS FOR USE

- 1 Insert the catheter in the pocket.
- 2 Adjust the size of the belt by superimposing both ends, using the specific velcro closing system.

- 3 Cut the ends of the cotton belt in excess.



- 4 Regulate the elastic tension and cut the elastic end in excess.



CLASSIFICATION	• CLASS I MEDICAL DEVICE
BIOCOMPATIBLE	• ACCORDING TO UNI EN ISO 10993 STANDARDS
STERILIZATION	• NOT STERILE
EXPIRATION	• THE DEVICE DOES NOT EXPIRE
REUSE	• REUSABLE 2/3 TIMES AFTER HAND WASHING AT 30°C MAX