

EXIT-POCKET® WP

Description

EXIT-POCKET WP is a waterproof pocket closed on four sides

Indications

Containment of catheter lumens during patient's hygiene procedures (shower and bath) and during water activities (diving, sea, swimming pool).


Purpose

Designed to protect the exit-site from water contact. The device is completely waterproof: if well-positioned it protects the exit-site from contact with water preventing the possibility of microbial contamination.

Benefits

- " Quick and easy placement.
- " Helps to improve the quality of life of catheterized patients.
- " Easy and fast removal.

Patented

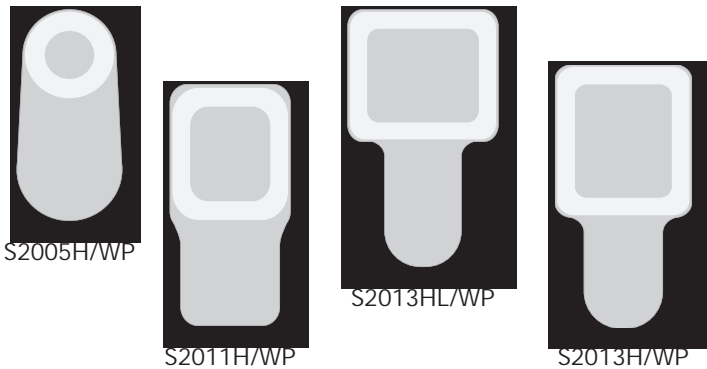

References


DDI/DE/FR/IT/PT

CODE	SIZE (cm)			INDICATION	PCS / BOX
	Hole	H	L		
S2005H/WP	ø 5	11	22	Bath and Shower	50
S2011H/WP	10 x 9	13	26	Bath and Shower	50
S2013HL/WP	13 x 11	17,6	30	Bath and Shower	50
S2013H/WP	11 x 13	15,6	30	Bath and Shower	50

Waterproof pocket for catheter protection

Directions for use

1. Cover the catheter lumen(s) with the device.
2. Remove the protective sheet from the adhesive.
3. Place the lower margin of the adhesive hole in correspondence with the catheter, avoiding wrinkles or airlocks.
4. Take shower or bath paying attention not to rub the area near the adhesive.
5. Remove gently from the skin a small strip of adhesive starting from the lower part. Pull carefully with small movements from bottom to top.

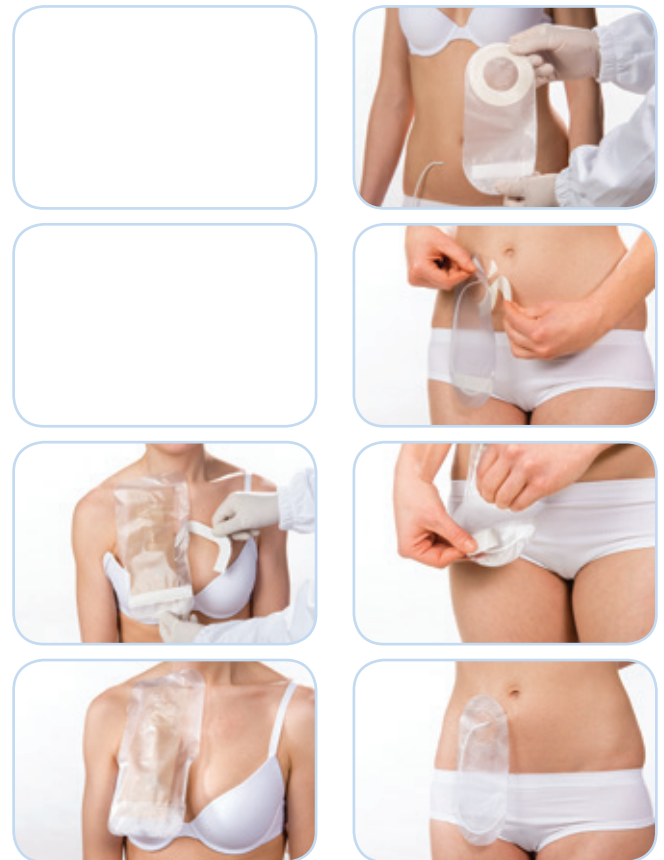

CLASSIFICATION • CLASS I MEDICAL DEVICE (EUROPEAN COMMUNITY)
 BIOCOMPATIBLE • ACCORDING TO UNI EN ISO 10993 STANDARDS
 STERILIZATION • NOT STERILE
 EXPIRATION • 2 YEARS

⚠️ ⓧ • SINGLE-USE