


Praxiject™ Saline

0.9% NaCl Prefilled Syringes

WARNINGS AND PRECAUTIONS :


- Use aseptic technique.
- Single use only. Do not reuse.
- For IV flush only.
- Do not use if package is opened or damaged.
- Do not use if the cap/adaptor on the syringe is not intact.
- Do not resterilise.
- Sterile unless package opened or damaged.

Usage Guidelines

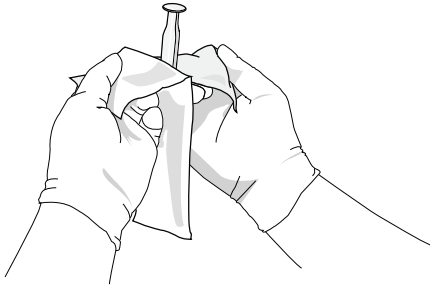


1

Aseptically open package containing Praxiject™ Saline prefilled syringe*. Visually inspect the contents of each prefilled syringe for clarity, particulate matter, precipitate, discoloration and leakage prior to use. Do not use if any of the aforementioned is observed.

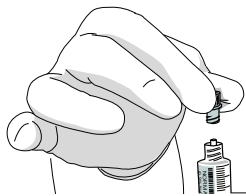

2

Remove and discard syringe cap.

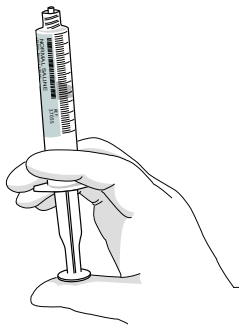

3

Expel air from syringe. Do not allow air to be trapped in fluid path. Use only with compatible luer lock connectors.



4

Following the institution's policy, instill the required volume of Praxiject™ solution.



5

Remove and discard unused portions and empty syringes according to the institution's biohazardous waste policy.

* Praxiject™ products are sterile field compatible