



Made in Italy by



EMODIAL



BETAFIX[®]

Fistula Bandage



www.valiantmedical.co.uk

+44(0)330 303 9090

info@valiantmedical.co.uk

VM-BETAFIX-BROCHURE-04/21



BETAFIX®

DESCRIPTION

Elastic fistula bandage provided with a two layer highly absorbent pad and adhesive for a better tightness. Also suitable for patients who require intravenous treatment and blood transfusion.

INTENDED USE

Post dialysis dressing to stop the bleeding after the removal of the fistula needles.

PURPOSE

Designed to stop bleeding and to reduce the hemorrhagic risk, thanks to the compression on the needle holes, also working as a barrier against the risk of infections. The non-adherent, biocompatible layer on the pad doesn't stick to the skin **avoiding the clot removal and the risk of bleeding**. The cotton bandage avoids the sensitization due to the glue of adhesive.



REFERENCES

Data and technical specifications can be changed without notice.

CODE	BANDAGE (cm)		PAD (cm)	PCS / PACK	PACKS / BOX
	L	H			
S0014	50 ± 10%	6	3.5 x 4.5	2	300
S0029	50 ± 10%	6	∅ 2.6	1	500
S0029/A	55 ± 10%	6	∅ 2.6	1	400
S0030	50 ± 10%	6	∅ 2.6	2	300
S0030/A	55 ± 10%	6	∅ 2.6	2	300
S0030/R	25 ± 10%	6	∅ 2.6	2	300
S0041	50 ± 10%	6	∅ 3.5	2	300

Fistula Bandage

DIRECTIONS FOR USE

-  Unroll the first part of the bandage containing the pad.
-  Put the pad over the needle site.
-  Tighten bandage sufficiently to stop bleeding.
-  Remove the protective film and fix the bandage with the adhesive.

CLASSIFICATION • CLASS I MEDICAL DEVICE
BIOCOMPATIBLE • ACCORDING TO UNI EN ISO 10993 STANDARDS
STERILIZATION • GAMMA RAYS
EXPIRATION • 5 YEARS

  • SINGLE-USE