



Made in Italy by



EMODIAL

CE
0373

SOVAN[®]

Fistula Dressing



www.valiantmedical.co.uk

+44(0)330 303 9090

info@valiantmedical.co.uk

VM-SOVAN-BROCHURE-04/21



DESCRIPTION

SOVAN is a highly absorbing two layers hemostatic pad set on hypoallergenic, air permeable adhesive. Also suitable for patients who require intravenous treatment and blood transfusion.

INTENDED USE

Compressive hemostatic action, specific for post dialysis arteriovenous access care. To be used for the hemostasis after the removal of the fistula needles.

PURPOSE

Designed to stop bleeding and to reduce the hemorrhagic risk, thanks to the compression on the needle holes, also working as a barrier against the risk of infections.

The non-adherent, biocompatible layer on the pad doesn't stick to the skin **avoiding the clot removal and the risk of re-bleeding.**

The adhesive helps to keep the pad in position.



DIRECTIONS FOR USE



1 Remove protective paper and fix the first half of the dressing to the skin.



2 Put the pad over the needle site, press and remove needle.



3 Remove the second half of the paper and fix tightly the other half of the dressing to the skin. Hold in place until hemostasis is achieved.

REFERENCES

Data and technical specifications can be changed without notice.

CODE	PLASTER (cm)	PAD (cm)		ACTIVE SURFACE PLASTER AREA	PCS / PACK	PACKS / BOX
		Size ϕ	Thickness			
S0074	15 x 5	2.6	0.6 \pm 1	75 cm ²	1	500
S0075	15 x 5	2.6	0.6 \pm 1	75 cm ²	2	500
S0079	10 x 5	2.6	0.6 \pm 1	50 cm ²	2	500
S0079-1	10 x 5	2.6	0.6 \pm 1	50 cm ²	1	700

CLASSIFICATION • CLASS I MEDICAL DEVICE
BIOCOMPATIBLE • ACCORDING TO UNI EN ISO 10993 STANDARDS
STERILIZATION • GAMMA RAYS
EXPIRATION • 5 YEARS



• SINGLE-USE